MAMMOOTH

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Designed for perfect sterilization of various materials, thanks to the combination of vacuum, temperature and drying.

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Preheating excluded this can vary from 20 to 25 minutes. As to Superior B 23 the total time is 15 more.

Superior B It. 6

Norme EN-13060

Materiale e Strumenti <i>Materials and Instruments</i>	Programma <i>Program</i>	Fasi di Vuoto <i>Vacuum Phases</i>	Tempo Totale <i>Total Time</i>
Cavi delicati e cavi inossidabili (imbustati) <i>Delicate hollow and rustproof hollow (wrapped)</i>	121° cavi imbustati <i>Hollow wrapped</i>	4	28 minuti <i>minutes</i>
Cavi inossidabili (imbustati) <i>Rustproof hollow (wrapped)</i>	134° cavi imbustati <i>Hollow wrapped</i>	4	15 minuti <i>minutes</i>
Cavi delicati e cavi inossidabili (non imbustati) <i>Delicate hollow and rustproof hollow (unwrapped)</i>	121° cavi aperti <i>Open hollow</i>	4	20 minuti <i>minutes</i>
Cavi in gomma e inossidabili (non imbustati) <i>Hollow made of rubber and stainless steel (unwrapped)</i>	134° cavi aperti <i>Open hollow</i>	4	7 minuti <i>minutes</i>
Solidi e cavi inossidabili (imbustati) <i>Rustproof solid and hollow (wrapped)</i>	134° prion <i>Prion</i>	4	30 minuti <i>minutes</i>
Delicati, cavi e porosi (imbustati) <i>Delicate, hollow and porous (wrapped)</i>	121° porosi <i>Porous</i>	4	33 minuti <i>minutes</i>
Solidi inossidabili, cavi e porosi (imbustati) <i>Solid rustproof, hollow, and porous (wrapped)</i>	134° porosi <i>Porous</i>	4	20 minuti <i>minutes</i>
Solidi in gomma e delicati (non imbustati) <i>Solid made of rubber and delicate (unwrapped)</i>	121° solidi aperti <i>Solid Open</i>	2	20 minuti <i>minutes</i>
Solidi inossidabili (non imbustati) <i>Solid rubber (unwrapped)</i>	134° solidi aperti <i>Solid Open</i>	2	7 minuti <i>minutes</i>
Solidi inossidabili (imbustati) <i>Rustproof solid (wrapped)</i>	134° solidi imbustati <i>Solid wrapped</i>	2	15 minuti <i>minutes</i>
Ciclo test per Helix e Bowie & Dick <i>Test Cycle for Helix and Bowie & Dick</i>	134° Helix/B&D	4	6 minuti <i>minutes</i>
Ciclo test per Vuoto <i>Test Cycle for vacuum</i>	<40° Vacuum Test	1	15 minuti <i>minutes</i>

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Preheating excluded this can vary from 8 to 12 minutes.

Superior S It. 18 - 23

Norme EN-13060

Materiale e Strumenti Materials and Instruments

Cavi delicati e cavi inossidabili (imbustati)
Delicate hollow and rustproof hollow (wrapped)

Programma Program

121° cavi imbustati
Hollow wrapped

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2

Tempo Totale Total Time

30 minuti
minutes

Cavi inossidabili (imbustati)
Rustproof hollow (wrapped)

134° cavi imbustati
Hollow wrapped

2

22 minuti
minutes

Solidi in gomma e solidi delicati (imbustati)
Solid made of rubber and delicate solids (wrapped)

121° solidi imbustati
Solid wrapped

2

28 minuti
minutes

Solidi inossidabili (imbustati)
Rustproof solid (wrapped)

134° solidi imbustati
Solid wrapped

2

15 minuti
minutes

Solidi e cavi inossidabili (imbustati)
Rustproof solid and hollow (wrapped)

134° prion
Prion

2

35 minuti
minutes

Delicati, cavi e piccoli carichi porosi (imbustati)
Delicate, hollow and porous (wrapped)

121° porosi
Porous

2

33 minuti
minutes

Solidi inossidabili, cavi e piccoli carichi porosi (imbustati)
Solid rustproof, hollow, and porous loaded small(wrapped)

134° porosi
Porous

2

20 minuti
minutes

Solidi in gomma e delicati (non imbustati)
Solid made of rubber and delicate (unwrapped)

121° rapido
Rapid

2

23 minuti
minutes

Solidi inossidabili (non imbustati)
Solid rubber (unwrapped)

134° rapido
Rapid

2

10 minuti
minutes

Cavi in gomma e inossidabili (non imbustati)
Hollow made of rubber and stainless steel (unwrapped)

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*Preheating excluded this can vary from 20 to 25 minutes.
As to Superior S 23 the total time is 15 more.*

Superior N It. 18 - 23

Norme EN-13060

Materiale e Strumenti Materials and Instruments

Solidi in gomma e solidi delicati (con asciugatura lunga)
Solid made of rubber and delicate solid (with prolonged drying)

Programma Program

121° solidi aperti
Solid Open

Tempo Totale Total Time

33 minuti
minutes

Solidi in gomma e solidi metallici (con asciugatura lunga)
Solid made of rubber and solid metal (with prolonged drying)

134° solidi aperti
Solid Open

21 minuti
minutes

Solidi inossidabili e strumenti metallici
Solid stainless steel and metal instruments

134° prion
Prion

35 minuti
minutes

Solidi in gomma e solidi delicati (con asciugatura corta)
Solid made of rubber and delicate solid (with brief drying)

121° rapido
Rapid

23 minuti
minutes

Solidi in gomma e solidi delicati (con asciugatura corta)
Solid made of rubber and delicate solid (with brief drying)

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*Preheating excluded this can vary from 20 to 25 minutes.
As to Superior N 23 the total time is 15 more.*

Light It. 18 - 23

Light It. 6

Materiale e Strumenti Materials and Instruments

Solidi in gomma e solidi delicati
Solid made of rubber and delicate solid (wrapped drying)

Programma Program

121° long dry

Tempo Totale It. 18 - 23 Total Time It. 18 - 23

33 minuti
minutes

Tempo Totale It. 6 Total Time It. 6

28 minuti
minutes

Solidi in gomma e solidi metallici
Solid made of rubber and solid metal (unwrapped)

121° short dry

27 minuti
minutes

23 minuti
minutes

Solidi inossidabili
Solid stainless steel and metal instruments (wrapped)

134° long dry

26 minuti
minutes

19 minuti
minutes

Solidi inossidabili
Solid stainless steel and metal instruments (wrapped)

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*Preheating excluded this can vary from 15 to 20 minutes.
As to Light 23 the total time is 15 more.*

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Mammoth reserves the right to make
modifications without prior warning

	Superior B lt. 18 - 23		Superior B lt. 6	Superior S lt. 18 - 23		Superior N lt. 18 - 23		Light lt. 18 - 23		Light lt. 6	
Chiusura motorizzata <i>Motor-operated closure</i>	●		●	●		●		Meccanico <i>Mechanical</i>		Meccanico <i>Mechanical</i>	
Sicurezza blocco porta <i>Door lock safety</i>	Elettrica <i>Electrical</i>		Elettrica <i>Electrical</i>	Elettrica <i>Electrical</i>		Elettrica <i>Electrical</i>		Meccanica <i>Mechanical</i>		Meccanica <i>Mechanical</i>	
Tipo di vuoto <i>Type of Vacuum</i>	Frazionato <i>Fractionated</i>		Frazionato <i>Fractionated</i>	Pre e post vuoto <i>Pre and Post vacuum</i>		Termodinamico <i>Thermodynamic</i>		Termodinamico <i>Thermodynamic</i>		Termodinamico <i>Thermodynamic</i>	
Vaporizzatore istantaneo <i>Immediate steam generator</i>	●		●	●		●		—		—	
Programmi Test <i>Test Programs</i>	Bowie & Dick/Helix/Vacuum		Bowie & Dick/Helix/Vacuum	Bowie & Dick/Vacuum Test		—		—		—	
Stampante a bordo <i>Onboard Printer</i>	●		●	●		●		—		—	
Display <i>Display</i>	Grafico <i>Graphic</i>		Grafico <i>Graphic</i>	Grafico <i>Graphic</i>		Grafico <i>Graphic</i>		LCD		LCD	
MTS Memory Test System <i>M.T.S. Memory System Test</i>	●		●	●		●		—		—	
Collegamento a Datalogger <i>Connection to PC</i>	●		●	●		●		—		—	
Collegamento a depuratore <i>Connection to demineralizer</i>	●		—	●		—		—		—	
Ciclo notte <i>Night Cycle</i>	●		●	●		●		●		●	
Serbatoi acqua interni (carico e scarico) <i>Internal water tanks (loading and unloading)</i>	2		2	2		2		1		1	
Capacità camera in Lt. <i>Chamber volume in litres</i>	18	23	6	18	23	18	23	18	23	6	
N° cicli con un carico di acqua (indicativo) <i>No. of Cycles with one water loading (indicative)</i>	8	6	18	10	8	10	8	16	12	10	12
Massa sterilizzabile imbustata in Kg. <i>Wrapped sterilizable weight in kg</i>	3	4,5	1	3	4,5	—		—		—	
Massa sterilizzabile libera in Kg. <i>Unwrapped sterilizable weight in kg</i>	4	6	1,8	4	6	4	6	4	6	2	
Materiale poroso in Kg. <i>Porous material in kg</i>	1	1,5	0,5	0,5	1	—		—		—	
Peso macchina a vuoto in Kg. <i>Empty machine weight in kg</i>	56	60	40	45	50	43	48	40	45	30	
Misure esterne - <i>Outer measurement:</i>											
Profondità - <i>Depth mm</i>	610	695	610	610	695	610	695	610	695		
Larghezza - <i>Width mm</i>	505	505	445	445	445	445	445	445	445		
Altezza - <i>Height mm</i>	400	400	345	400	400	400	400	400	400		
Tensione di alimentazione <i>Supply voltage</i>	220-240 V										
Frequenza di rete <i>Mains frequency</i>	50-60 Hz										
Potenza nominale <i>Rated power</i>	2400 W		2500 W	2400 W		2400 W		1800 W		1800 W	

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Tests for Autoclaves

*Wide range of tests:
 Helix Test to check sterilization inside hollow instruments
 Bowie & Dick Test: to check sterilization in porous materials
 Chemical Indicators: to check each sterilization cycle
 Spores: for the biological control of sterilization.*

Perfect Water
 Distillatore elettrico
Perfect Water
Electrical water distiller

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