

HMCE **"Clean One2"** Washer Disinfector

Automatically Stores, Cleans and Prepares soiled instruments ready for final sterilisation without any handling!

The HMCE "Clean One2" is an automatic "all in one" chamber for pre-disinfection, cleaning, rinsing and drying, prior to sterilisation

Clean One2 is an ergonomic and energy efficient design that uses Ultrasonic cleaning technology which is renown for its excellent effectiveness

Its many benefits include preservation of instruments and the elimination of infection through injury as soiled instruments never require handling



Step1 – Storage:

- Pre Disinfects soiled instruments by immersing in a bath of measured chemical solution*

Step2 – Cleaning:

- Ultrasonic cleaning technology which penetrates even the most difficult and inaccessible areas

Step3 – Rinsing:

- Ultrasonic rinsing with clean water provides instrument preservation by avoiding typical corrosion from chemical residue

Step4 – Drying:

- Tangential 80°C hot air flow provides even and consistent drying, even for an irregular arranged workload



Soiled Instruments are simply placed directly into the "Clean One2" immediately after patient treatment

Instruments are stored in a bath of Pre-disinfection solution prior to a process of Ultrasonic cleaning, rinsing and drying and emerge ready of sterilisation

All stages of the full Pre-sterilisation cleaning process is completed automatically without any handling

Benefits:

- One single product for all stages of the pre-sterilisation routine
- Automatically complies with the "Dirty ⇌ Clean" procedures
- Soiled instruments never require handling therefore reduces infection risk
- Ultrasonic cleaning for improved preservation of instruments
- Frees up space

Visit our website @ www.dbdental.co.uk

HMCE “Clean One2” Washer Disinfector

HMCE specialise in Disinfection and Sterilisation products and are subjected to stringent testing to meet all the required EN manufacturing standards

In addition to the controls and testing carried out in HMCE's own workshops to meet these EU standards, HMCE products are accredited by Apave which assure their operational reliability.

Technical Specification

Specific Data	Clean One2 18ltr	Clean One2 27ltr
External dimensions (mm)	556 x 560 x 460mm	556 x 660 x 460mm
Receptacle	329 x 300 x 200mm	480 x 290 x 200mm
Power	600w	600w
Transducers	8	12
Weight	40kgs	
Supply	220v - 240v 50/60Hz	
Solution Reservoir	4.5ltrs	
Cycle	35mins - 50mins according to program Pre-disinfection Normal, Instruments loose Normal, Instruments in tapes Intensive, Instruments loose Intensive, Instruments in tapes	
Safety	Water inlet by solenoid valve Unlocked door detector Prompt for emptying	
Guarantee	2 Years	

Environmental and ecological concerns are also very much part of HMCE's design and manufacturing processes, from choice of raw materials and packaging which are recyclable to minimising energy consumption. HMCE also use metal in favour of plastic.

All products are fully guaranteed, certified and validated.

DBdentalequipment
Sales Office & Customer Services
Tel: 01484 401015,
Fax: 01484 401018
Email: sales@dbdental.co.uk



Visit our website @ www.dbdental.co.uk